


Basic Maintenance

STERLINK Mini 6M

6 Month Maintenance



6 Month

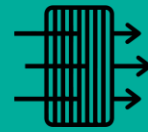

6 Month Filter



After installation
Change your **filters** every 6 months


Use 6 Month Maintenance Kit
to change **filters**

If you have any issues, send
an email to Plasmapp

Support@plasmapp.com
Tel : +1 (786) 218-7579


* Maintenance Kits

6M Filter Kit

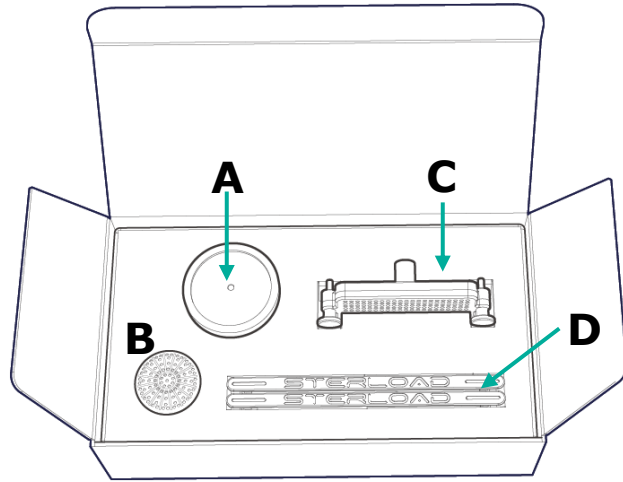

- A** HEPA Filter
- B** Chamber Filter
- C** Ozon Filter
(Two hand knobs)
- D** VC Cartridge

VC Cartridge
 Shelf life: None / Not included hydrogen peroxide
 Required a calibration every 6 months or when the installation place is changed

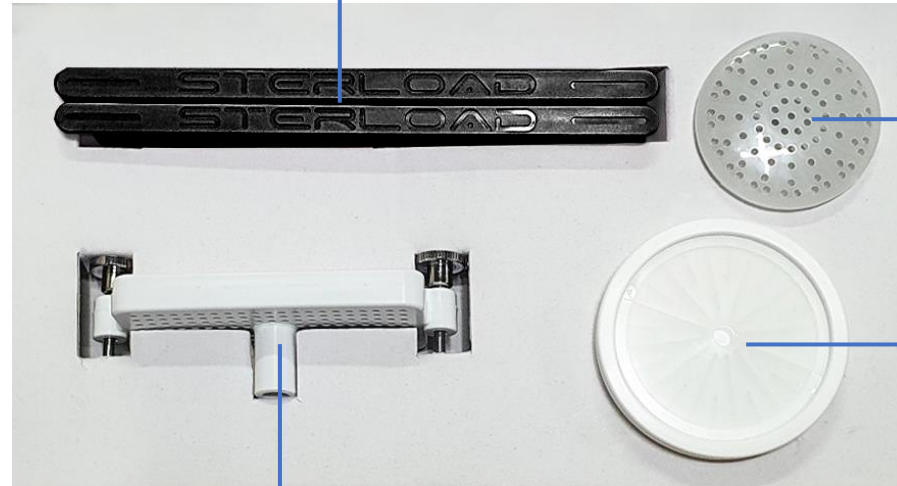


Chamber Hole Filter
 Shelf life: None
 Storage temperature:
 +5~80°C

HEPA Filter
 Storage temperature: +5~40°C

Ozon Filter
 Shelf life: None/Storage temperature: +5~80°C
 (Including two Ozone Filter Hand Knobs)

Filter Change Period

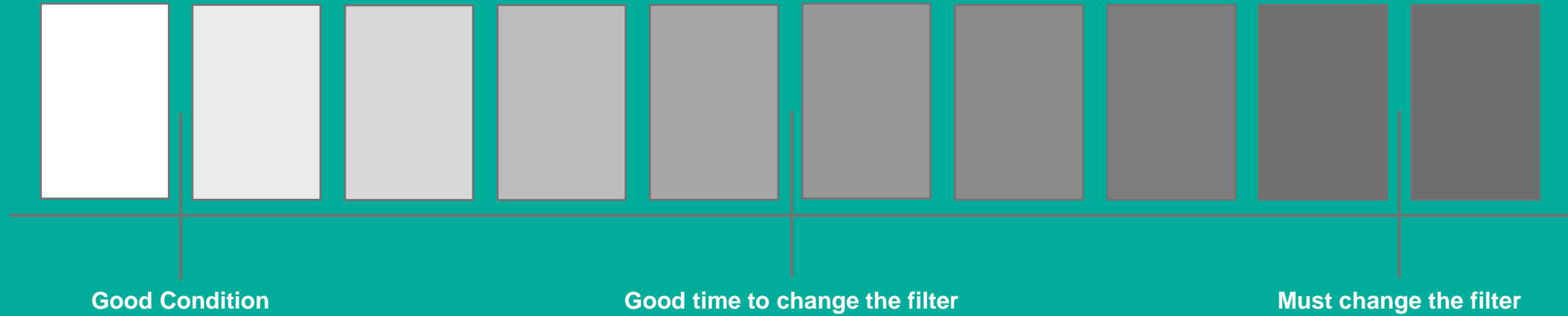

Maintenance

HEPA / Chamber Filter Change

HEPA Filter Change



1

Check the HEPA filter in the back of the product


2

Turn the HEPA filter counterclockwise and remove the used HEPA filter.

3

Unwrap the new HEPA filter first, fit into the hole, and turn the HEPA filter clockwise.

Chamber Filter Change

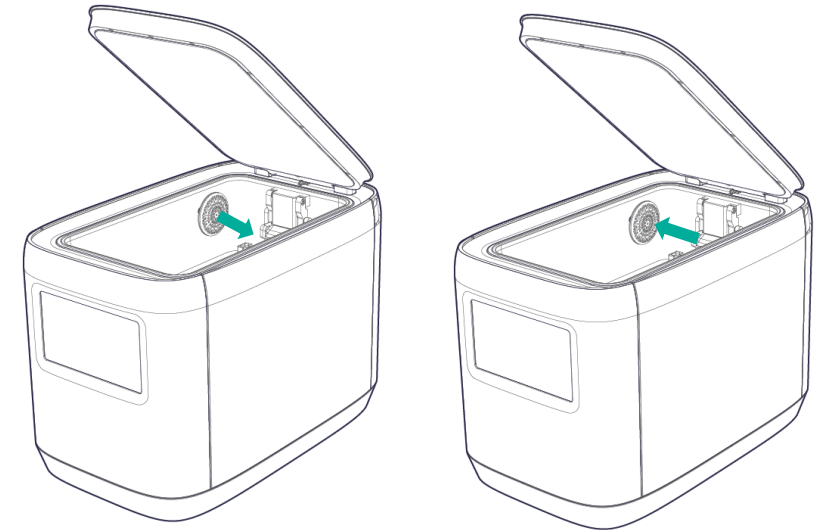

1


Pull up & remove the chamber filter in the chamber

2

Make the new chamber filter fit in the hole & Push to the end


Maintenance

Deodorizing & Ozon Filter Change

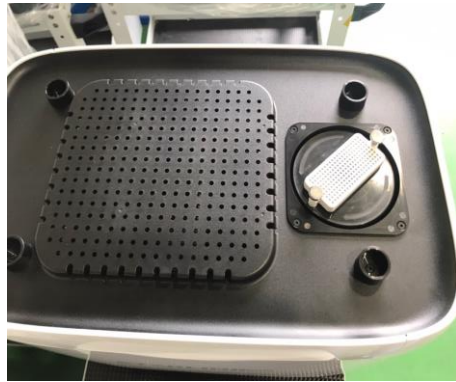


1

Remove the upper cover of MINI Beta

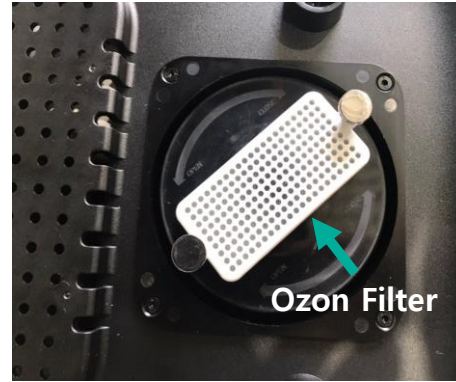


2

Turn the deodorizing filter & Ozon filter together counterclockwise and remove from the MINI

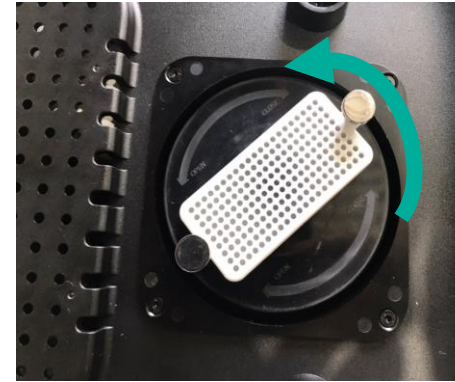

3

Fix the Ozon filter using hand knob on the deodorizing filter

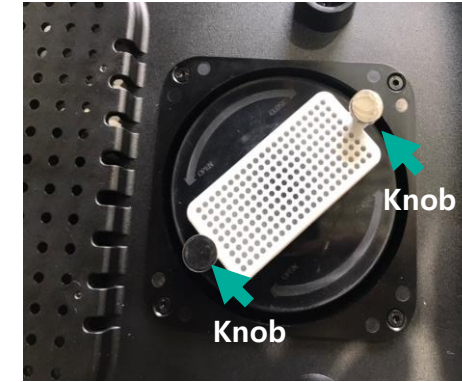

4

Put the deodorizing filter to the filter case hole and turn it clockwise for fixation