

Basic Maintenance

STERLINK plus 12M

12 Month Maintenance



1 Year



12 Month Filter



After installation
Change your **filters** every 6 months
Change **oil** every 12 months

Use 12 Month Maintenance Kit
to change **filters** and **oil**


If you have any issues, send
an email to Plasmapp

Support@plasmapp.com
Tel : +1 (786) 218-7579

Maintenance Kits


Filters & Oil Change

12M Filter Kit

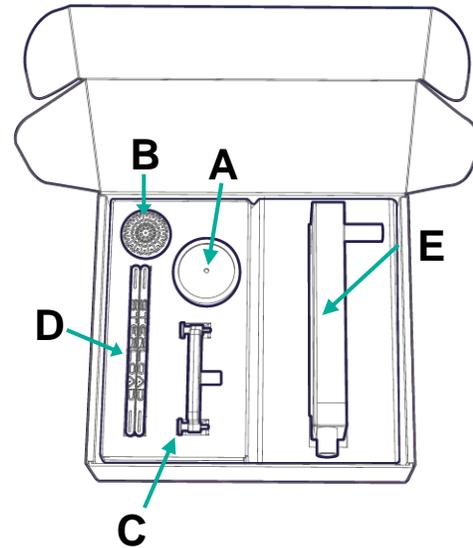


- A** HEPA Filter
- B** Chamber Filter
- C** Ozone Filter
(Two hand knobs)
- D** VC Cartridge
- E** Deodorizing Filter

12M Oil Kit

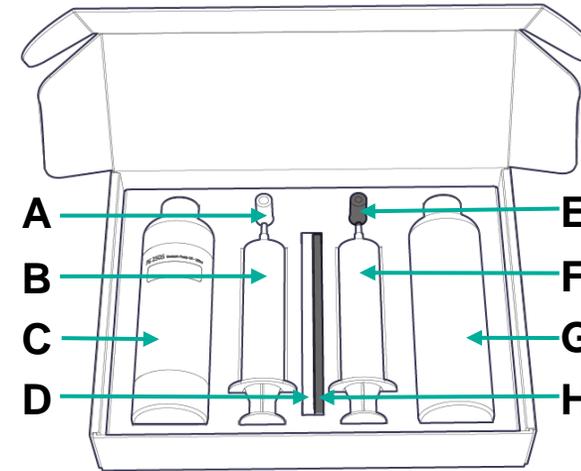


- A** New oil injection silicon adapter (white)
- B** New oil injection syringe
- C** New oil bottle
- D** New oil injection tube (transparent)
- E** Extracted oil injection silicon adapter (black)
- F** Extracted oil syringe
- G** Extracted oil bottle
- H** Waste oil extraction tube (black)

* Oil Change Kit

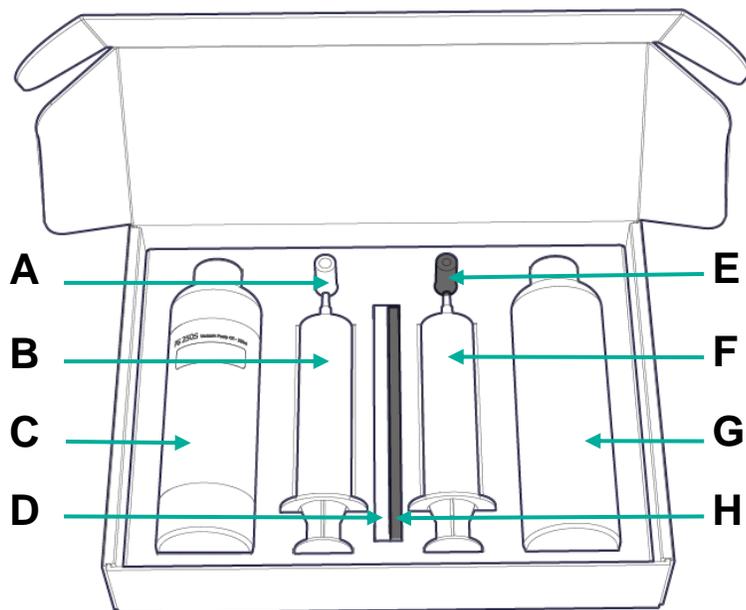


- A** New oil injection silicon adapter (white)
- B** New oil injection syringe
- C** New oil bottle
- D** New oil injection tube (transparent)
- E** Extracted oil injection silicon adapter (black)
- F** Extracted oil syringe
- G** Extracted oil bottle
- H** Waste oil extraction tube (black)

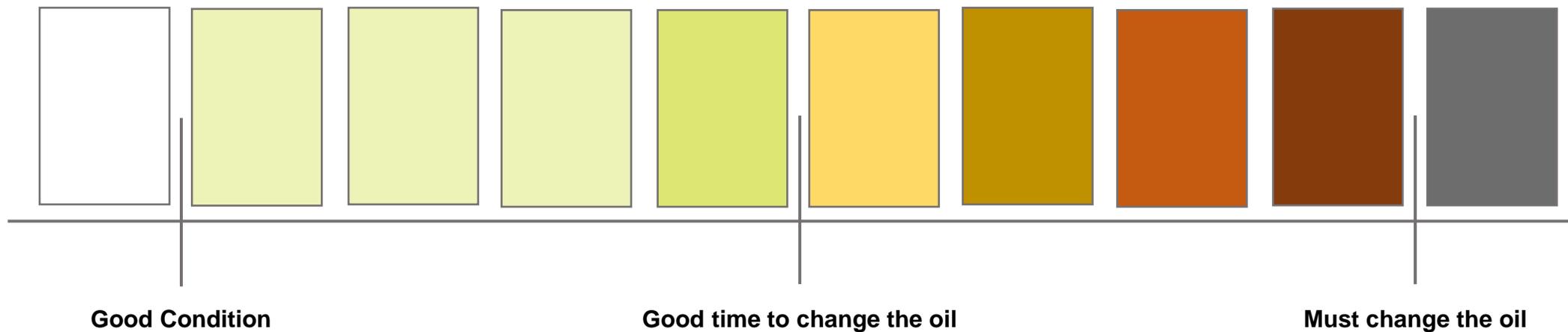


* Oil Change Using Oil Kit



* 12M Filter Kit

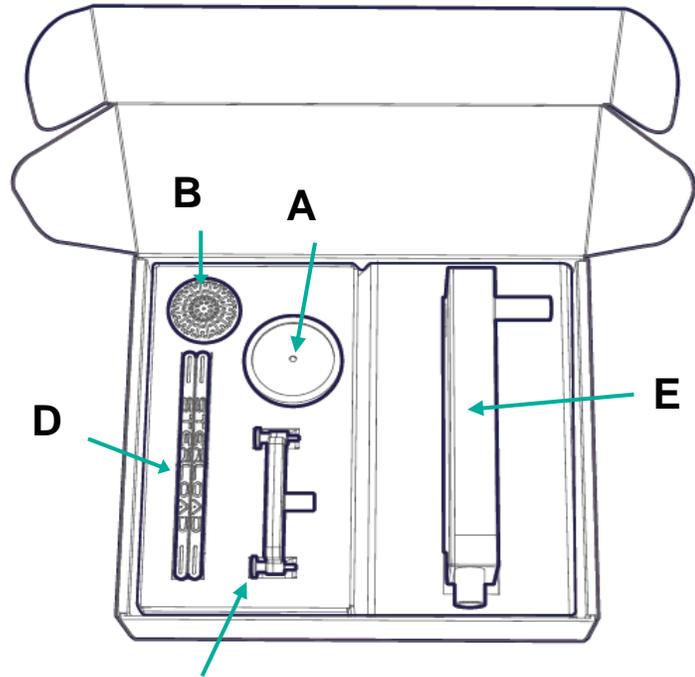

- A** HEPA Filter
- B** Chamber Filter
- C** Ozone Filter
(Two hand knobs)
- D** VC Cartridge
- E** Deodorizing Filter

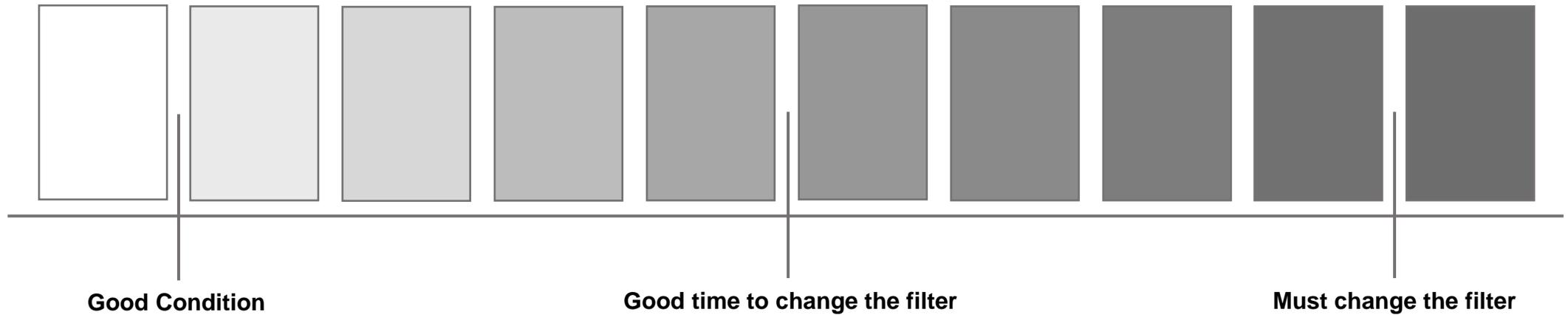

Filter Change Using Filter Kit


Changing HEPA Filter



1. Take out the new HEPA filter




2. Remove the old HEPA filter by turning counterclockwise

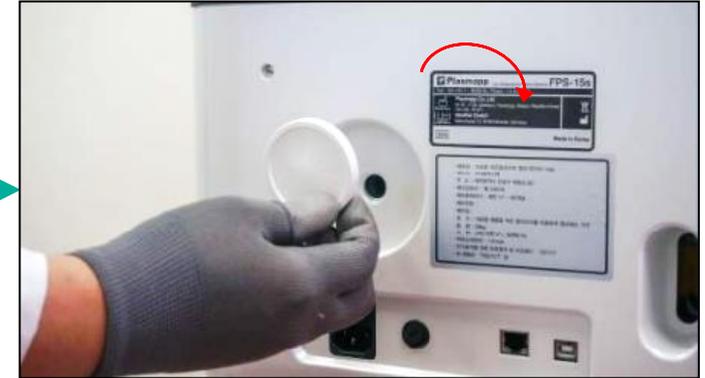

3. Replace it with new HEPA filter by turning clockwise

Changing Ozone Filter

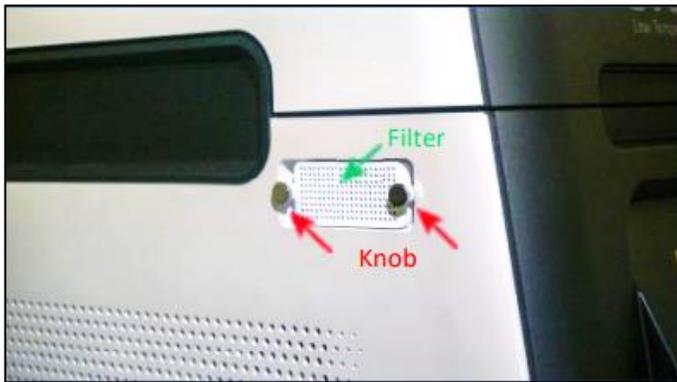

1. Ozone filter can be found on the left side of the product.



2. Remove the Knobs



3. Remove the Ozone filter and replace it with new Ozone filter