


Basic Maintenance

STERLINK plus 6M

6 Month Maintenance



6 Month

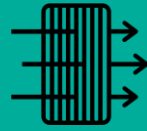

6 Month Filter



After installation
Change your **filters** every 6 months


Use 6 Month Maintenance Kit
to change **filters**

If you have any issues, send
an email to Plasmapp

Support@plasmapp.com
Tel : +1 (786) 218-7579


* Maintenance Kits

6M Filter Kit

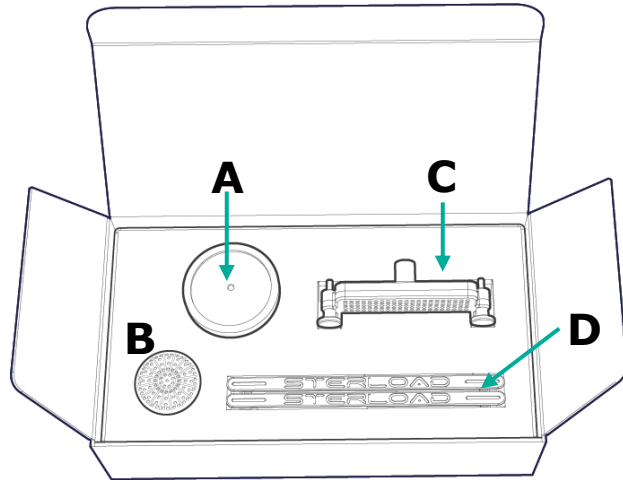

- A** HEPA Filter
- B** Chamber Filter
- C** Ozon Filter
(Two hand knobs)
- D** VC Cartridge

VC Cartridge
 Shelf life: None / Not included hydrogen peroxide
 Required a calibration every 6 months or when the installation place is changed

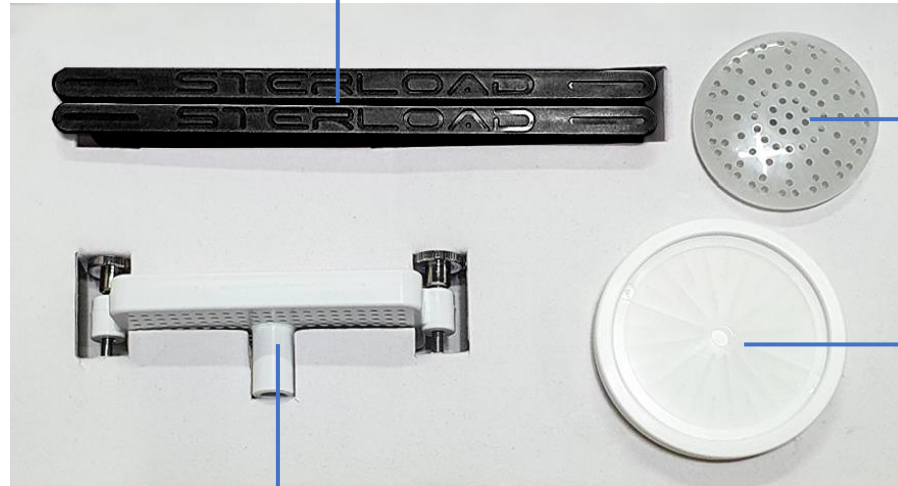


Chamber Hole Filter
 Shelf life: None
 Storage temperature:
 +5~80°C

HEPA Filter
 Storage temperature: +5~40°C

Ozon Filter
 Shelf life: None/Storage temperature: +5~80°C
 (Including two Ozone Filter Hand Knobs)

Filter Change Period

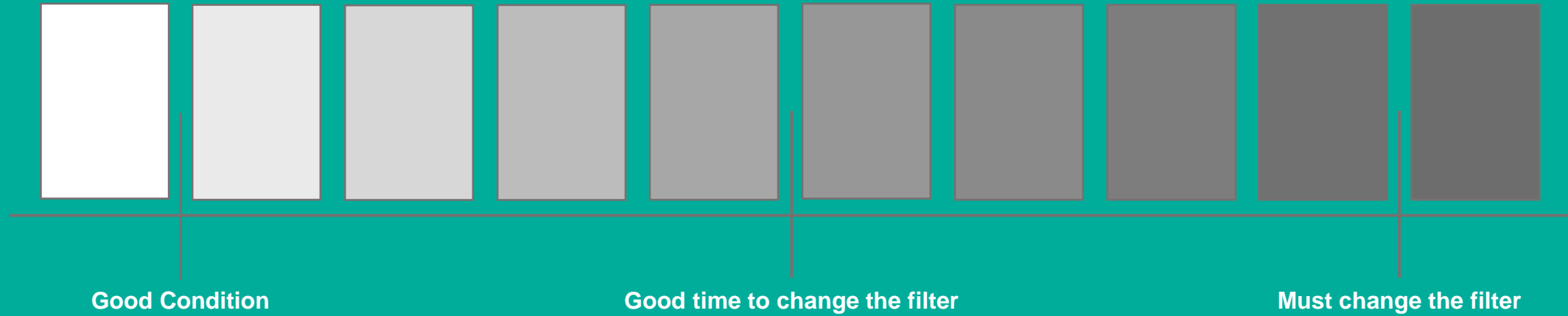

Filter Change Using Filter Kit


Changing HEPA Filter



1. Take out the new HEPA filter




2. Remove the old HEPA filter by turning counterclockwise

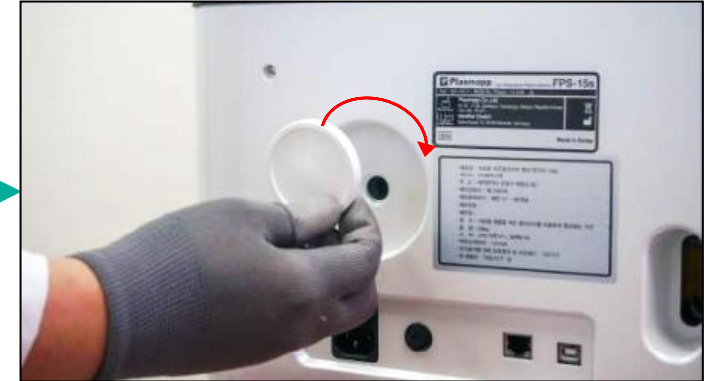

3. Replace it with new HEPA filter by turning clockwise

Changing Ozone Filter

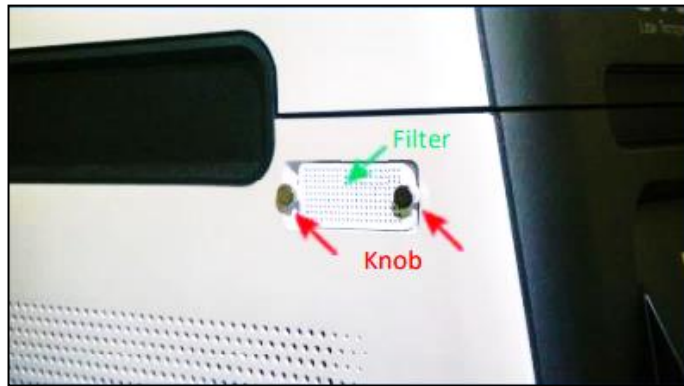

1. Ozone filter can be found on the left side of the product.



2. Remove the Knobs



3. Remove the Ozone filter and replace it with new Ozone filter